

## TANITA EU: Setting up email for sending reports in the TANITA PRO software

At this point you can make the appropriate settings to send a printout directly from the TANITA PRO software to the e-mail address stored for the respective customer.

The screenshot displays the 'E-Mail' configuration window in the TANITA PRO software. The window is divided into two main sections. The left section, titled 'SMTP settings', contains the following fields: 'Server' (smtp.web.de), 'Port' (587), 'User name' (username), 'Password' (masked with dots), a checked 'Use TLS/StartTLS' checkbox, and a 'Bcc' field. The right section shows a preview of an email. The 'From' field is 'emailaddress' and the 'Subject' is 'Your <Product> assessment'. The email body starts with 'Hello <Name>', followed by a line indicating that analysis results for a specific product and date are attached. It ends with 'Best regards, Your diagnostics team'. Below the preview, there are 'E-mail modules' with buttons for 'Name', 'Product', and 'Measurement date'. At the bottom of the window, there are three buttons: 'Check settings', 'Standard text', and 'Save'.

First configure your SMTP server under "SMTP Settings". The required data can be found in the following table:

Provider	Server	Port	TLS/StartTLS
Gmail	smtp.gmail.com	587	Yes
Outlook.com	smtp.live.com	587	Yes
Office365.com	smtp.office365.com	587	Yes
Yahoo Mail	smtp.mail.yahoo.com	587	Yes
Yahoo UK	smtp.mail.yahoo.co.uk	587	Yes
O2	smtp.o2online.de	587	Yes
AOL	smtp.aol.com	587	Yes
Hotmail	smtp.live.com	587	Yes
T-Online	securesmtp.t-online.de	587	Yes
1&1	smtp.1und1.de	587	Yes
Comcast	smtp.comcast.net	587	Yes
Mail.com	smtp.mail.com	587	Yes
Mail.de	smtp.mail.de	587	Yes
GMX.com	mail.gmx.net	587	Yes
Firemail.de	firemail.de	587	Yes
web.de	smtp.web.de	587	Yes
iCloud	smtp.web.de	587	Yes

If your provider is not listed, you can usually find the corresponding data in the settings of the provider's Web interface.

In the fields "Username" and "Password" you have to enter your login data.

If your username is also your e-mail address, please make sure to enter it completely, including the domain part. Check the box "Use TLS/StartTLS" according to the table or your provider's specifications. Another necessary field is the "sender". You must enter your complete e-mail address here, unless otherwise specified by the provider.

Single	Trend
<p>Hello &lt;Name&gt;, please find the results of your analysis with &lt;Product&gt; dated &lt;MeasurementDate&gt; attached.</p> <p>Best regards, Your diagnostics team</p>	

In this section you can store user-defined texts for the recipient of the e-mails you'd like to send later. You have the option of storing different texts for sending individual and progress result(s). However, the "subject" line always refers to both texts.

E-mail modules

Name	Product	Measurement...
Check settings		Standard text

Using the "e-mail modules" you have the possibility to dynamically adapt the text to the different products and customers. Pressing one of these buttons inserts the corresponding placeholder at the current position in the text. The placeholder "Name" is replaced by the first and last name of the patient when sending the e-mail.

The placeholder "Product" is replaced by the name of the product from which the email is sent. The placeholder "Measurement date" is only available for the individual result and is replaced by the date of the respective measurement. If the fields for the texts are empty when leaving the settings, they will automatically be filled with the standard texts stored by SportMed A.G. SA. Finally you have the possibility to test your settings with the button "Check settings". In the record archive of the respective product, you have the option of sending the selected measurement by clicking "Send e-mail" after successful set-up of the e-mail settings.