


TANITA PRO Software

How to: Delete all client profiles at once.

This step-by-step guide shows how to delete all client profiles at once.

Open the TANITA PRO Software.

Click on 'Clients':

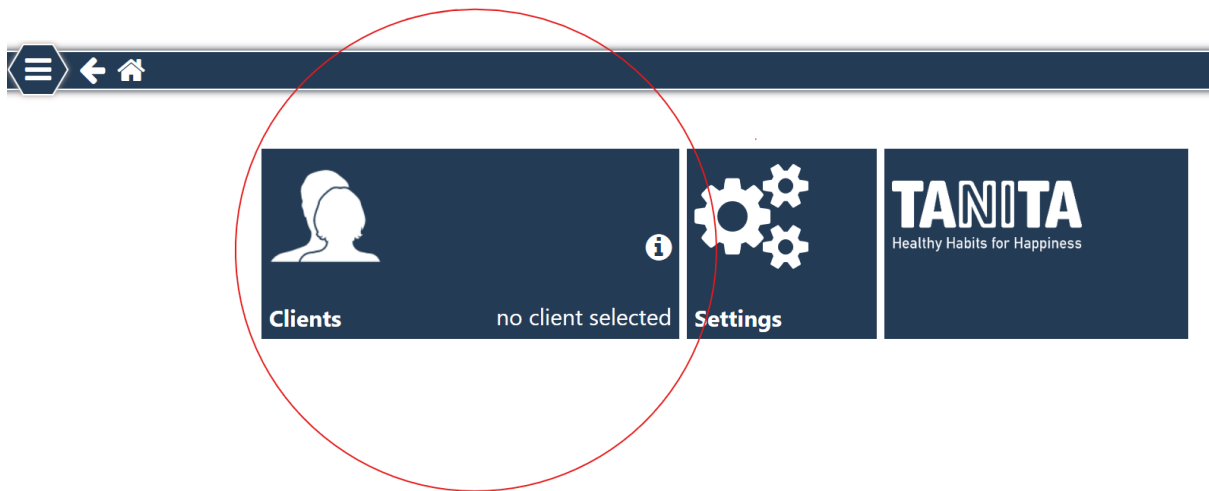

There you will see the whole list of client profiles that are currently active in the software:

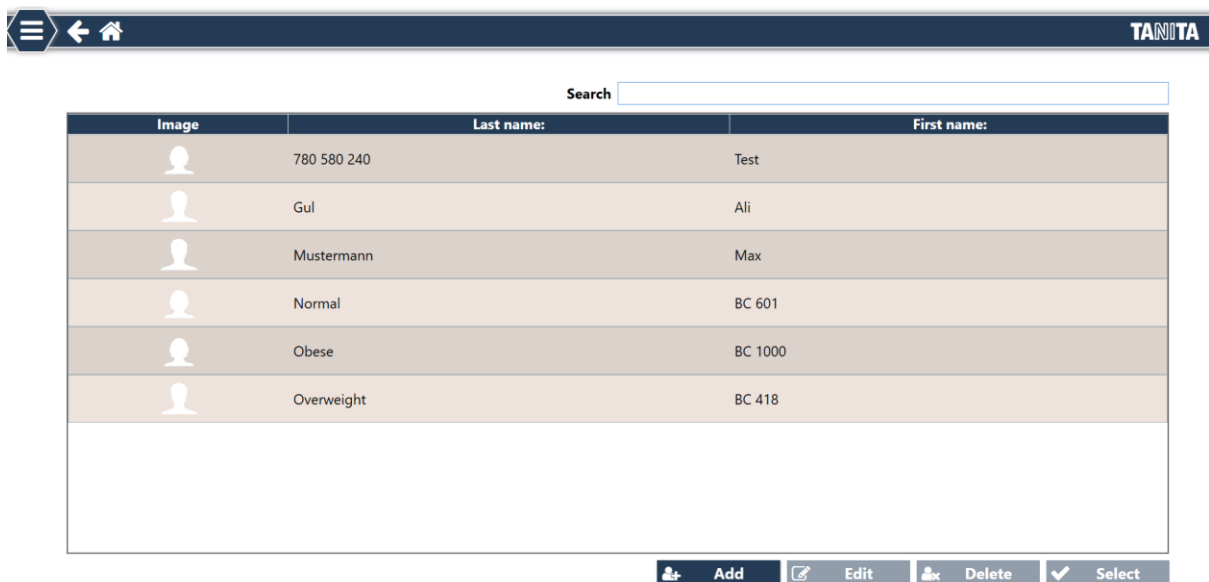















Image	Last name:	First name:
	780 580 240	Test
	Gul	Ali
	Mustermann	Max
	Normal	BC 601
	Obese	BC 1000
	Overweight	BC 418







 Add  Edit  Delete  Select





On the keyboard of your computer, now press the 'Ctrl' and 'a' key at the same time. This is the command to select all client profiles.

It will look like this:

TANIT

Search

Image	Last name:	First name:
	780 580 240	Test
	Gul	Ali
	Mustermann	Max
	Normal	BC 601
	Obese	BC 1000
	Overweight	BC 418

 Add  Edit  Delete  Select

Then click the 'Delete' button on the bottom, and all client profiles will be erased at the same moment:

