


TANITA EU: TANITA PRO software

How to import customer and measurement data into the TANITA PRO software from GMON database that is installed on a different computer / laptop

1. How to export the GMON data from the database to a file you can use for import

On the computer / laptop with TANITA PRO software, find and copy this whole folder:

C:\Program Files (x86)\SportMed\mobee360\Plugins\Tanita\GMONDataImporter

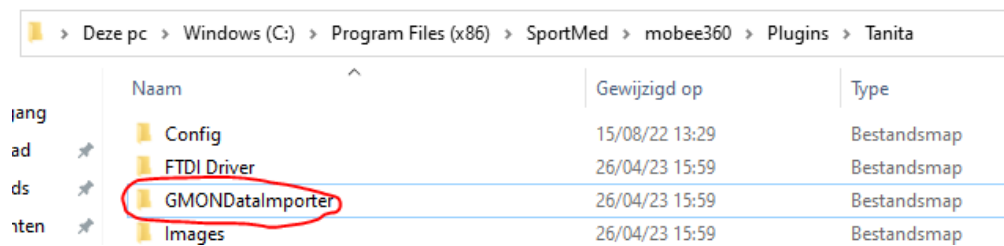


Copy, and paste the whole folder on an USB stick.


Now put the USB stick in your old laptop with GMON software installed on it.

Start the file GMONDataImporter.exe (🔗 GMONDataImporter) and the program window opens.


Choose “Work with data from GMON”:

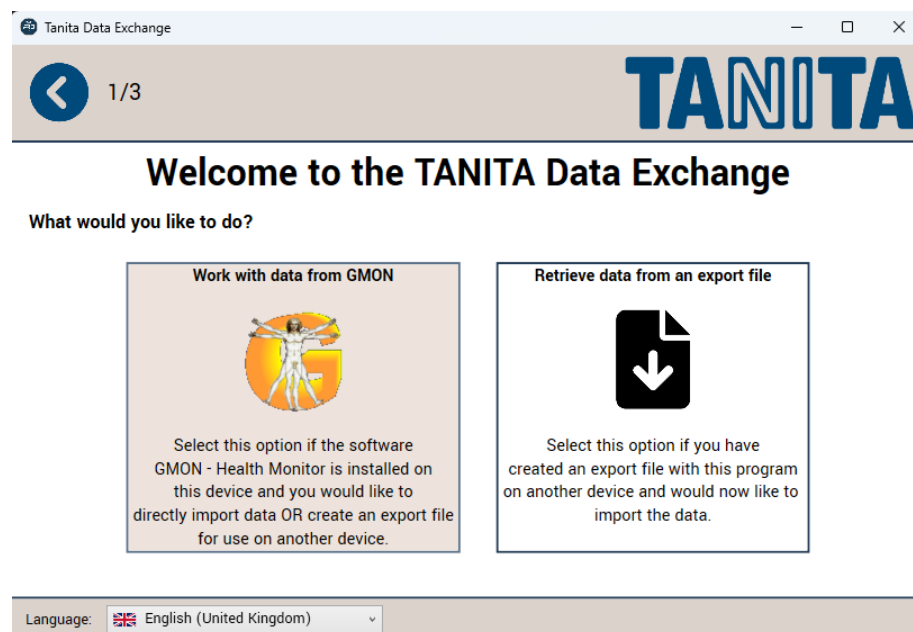

Choose “Into an export file”:

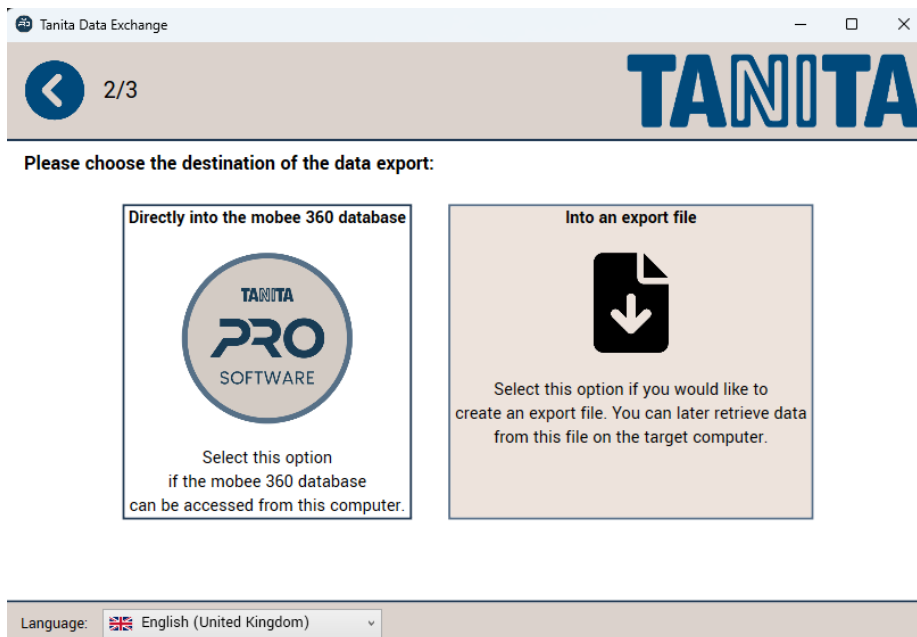


Select a location to save the file (like the USB stick) and click “Export data”:

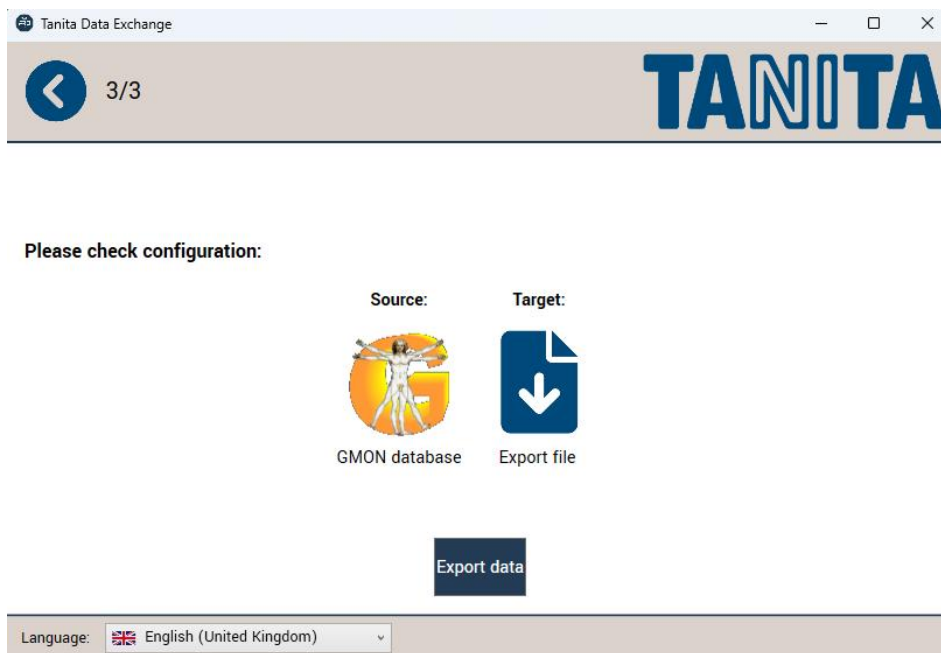


Put the USB stick in the new laptop and copy the file GMON_Export.xml that was just created from the computer/laptop with GMON on it to the computer with TANITA PRO on it.

2. Import the database export from GMON into the TANITA PRO software

Start the TANITA Data Exchange program again (from the USB stick).
Choose “Retrieve data from an export file”:

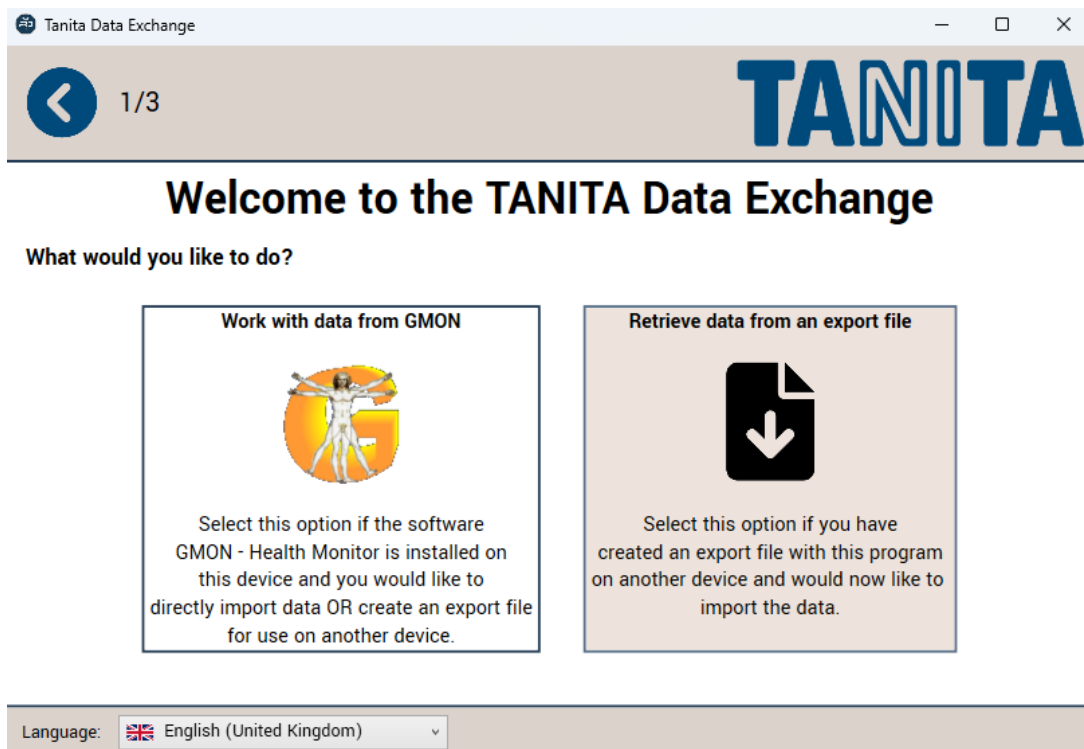

Browse to the file GMON_Export.xml and select it.
Then choose “Directly into the mobee360 database”:

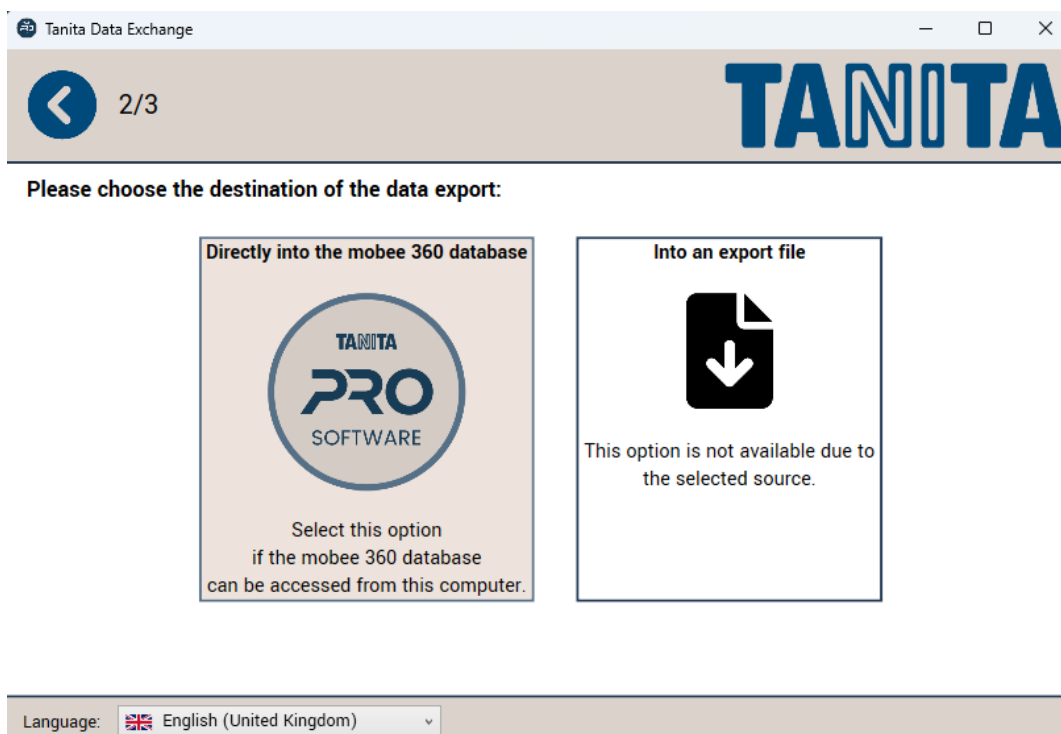



Choose “Import data”:

Tanita Data Exchange

3/3

TANITA

Please check configuration:

Source:	Target:
	
Export file	mobee 360 database

Import data

Language:  English (United Kingdom) v

Now the customer data and measurement data should be imported into the TANITA PRO software, and you can continue using that.