


How to remove the column from the MC-780MA P

- Switch off the MC-780MA, disconnect the power supply from the outlet and from the device.
- Remove all cables at the back of the display, such as the PC cable or printer cable.

1. Begin by removing all cable covers on the back of the column.

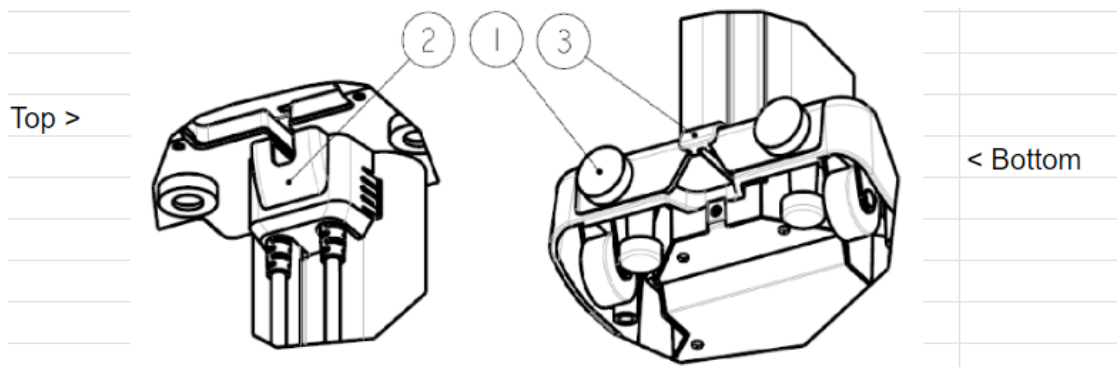

- Locate the plastic cable cover (marked "2" on the image) on the device. Press gently on both sides of the cover to click it off and pull it towards you to remove it.
 - Remove the plastic slot cover, located along the full length of the back of the column. Do this by sticking your finger behind it at the top (or bottom if there is more space) and pulling it towards you. Proceed carefully to avoid cracks in the slit cover.
 - Locate the rubber cable cover (marked "3" on the image) on the device. Hook your finger behind it and pull it out.
2. Loosen the screws under the screen. You can easily unscrew them by hand, without needing a screwdriver or special tools. Note that the screws do not come out completely, they are fixed in the holder with a washer (ring).
 3. Once the screws are undone, keep the device upright and gently pull the display towards you to release it from the holder. If pulling does not work, try pushing it in the opposite direction. The device has the flexibility to be adjusted based on user preference, so try both ways to ensure successful disassembly.
 4. Once the display is disconnected, remove the cables of the hand grips and the display from the column and from the and the hooks holding the cables in place, and place them in a safe place on the ground.
 5. After removing the display and hand grip cables from the column, you will see two more screws at the bottom of the column (marked "1" on the image). Loosen and remove these two screws by hand. Then gently pull the column towards you from the bottom. This will loosen the column and you can detach it from the device.

If you send the MC-780MA to TANITA EU for repair/calibration, please keep the column and its accessories at your location.